

# HOLTER REPORT

Patient: Test, Test

ID: 0000000001

DOB: 1/1/1900 Age: 111 Years

Sex: NA Scan No.:

Report Date: 3/9/2011, 8:34 AM

Test Date: 3/3/2011, 12:36 PM

Referring Physician: None Listed

Interpreting Physician: None Listed

Hook-Up: None Listed

Scanned By: Neil Moskowitz, C.V.T.

Site ID: Test

Recorder SN: 100909

## Indications:

Palpitations,

## Medications:

MVI,

## Summary:

The total recording time was: 48:00 hours. Total Beats = 231034.

The total time analyzed was: 46:40 hours.

Heart Rate: MAX = 139 at 10:08:45 AM(3), MIN = 63 at 5:09:30 PM(1), AVG = 80

There were 0 pause(s) > 2.5 seconds. The longest pause = 0.00 sec. at 12:36 PM(1).

VENTRICULAR SUMMARY				SUPRAVENTRICULAR SUMMARY			
Total :	0	Bigeminy Events :	0	Total :	22		
Isolated :	0	Bigeminy Beats :	0	Isolated :	12		
Pairs :	0	Trigeminy Events :	0	Pairs :	5		
R on T :	0	Trigeminy Beats :	0	Aberrant :	0		
Runs :	0	Quadrigeminy Events :	0	Runs :	0		
Run Beats :	0	Quadrigeminy Beats :	0	Run Beats :	0		

Longest: VE run = 0 beats at 12:36:23 PM(1)-SVE run = 0 beats at 12:36:23 PM(1)

Fastest: VE run = 0 BPM at 12:36:23 PM(1) - SVE run = 0 BPM at 12:36:23 PM(1)

## Conclusions:

Excellent Technical Quality. Channel 1 represents Lead II, Channel 3 represents Lead V1.

Rhythm is Sinus Rhythm (intermittent Sinus Arrhythmia recorded). Average HR is 80 bpm, minimum HR is 63 bpm, and maximum HR is 139 bpm.

No significant pauses or AV Block documented. Normal PR interval, QRS duration, and QT interval at varying heart rates. Longest R-R interval is .98 seconds.

Isolated APDs and Atrial Couplets documented. No complex atrial arrhythmia seen.

No complex ventricular arrhythmia seen.

Diary entries reported. Symptoms of 'Fast Heart beat/dizzy/sob' do not have arrhythmia correlate.

Physician's Signature \_\_\_\_\_

Date

5:30:23 PM(1)	HR: 86	Fast Heart Beat, Dizzy, SOB
9:36:55 AM(2)	HR: 72	SVPB Pair
9:38:25 AM(2)	HR: 80	SVPB Pair
9:38:30 AM(2)	HR: 82	Isolated SVPB
10:00:23 AM(2)	HR: 66	Fast Heart Beat, Dizzy, SOB
10:10:12 AM(2)	HR: 64	Minimum HR
11:00:23 AM(2)	HR: 68	Fast Heart Beat, Dizzy, SOB
3:19:47 PM(2)	HR: 63	Minimum HR
4:32:38 PM(2)	HR: 76	Isolated SVPB
7:01:20 PM(2)	HR: 133	Maximum HR
9:30:02 PM(2)	HR: 77	Sinus Rhythm
11:30:23 PM(2)	HR: 78	Fast Heart Beat, Dizzy, SOB
10:08:45 AM(3)	HR: 139	Maximum HR



5:30:23 PM(1)

.71

.68

.66

.66

**Fast Heart Beat, Dizzy, SOB**

.67

.66

.65

.64

**HR: 86**

.63

.63



9:36:55 AM(2)

.87

.85

.90

.54

**SVPB Pair**


.52


1.219

.97

**HR: 72**


.94

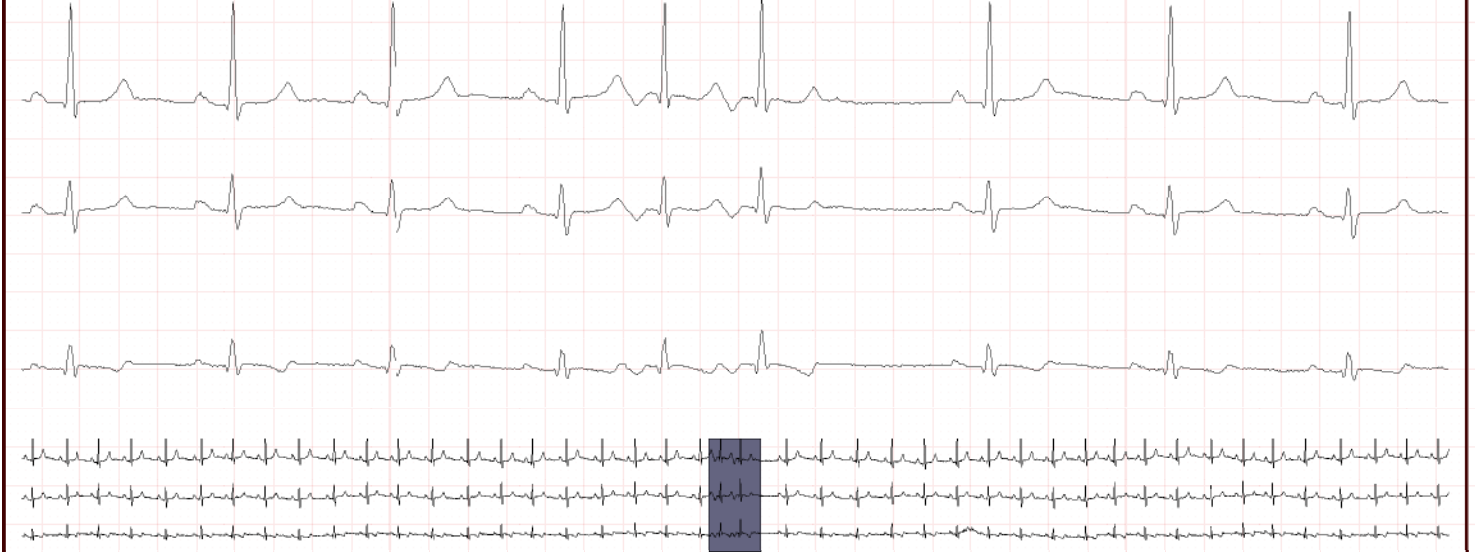

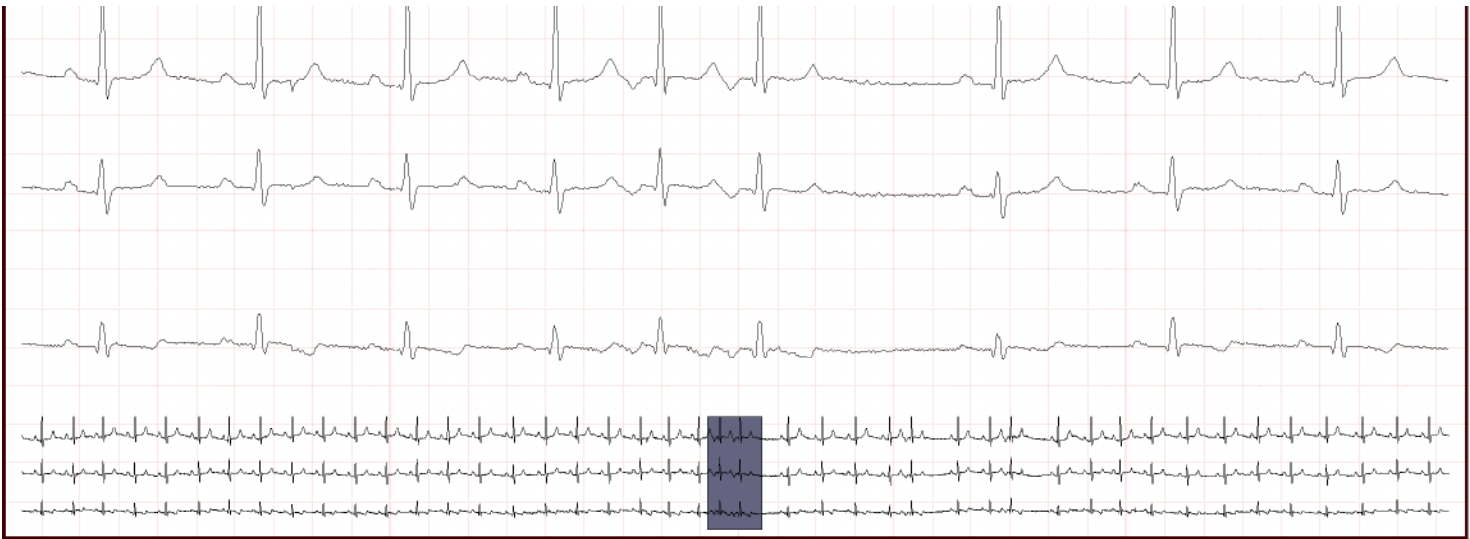


9:38:30 AM(2)


.92 .88 .91 .57 **Isolated SVPB** 1.226 .88 .56 **HR: 82** 1.250

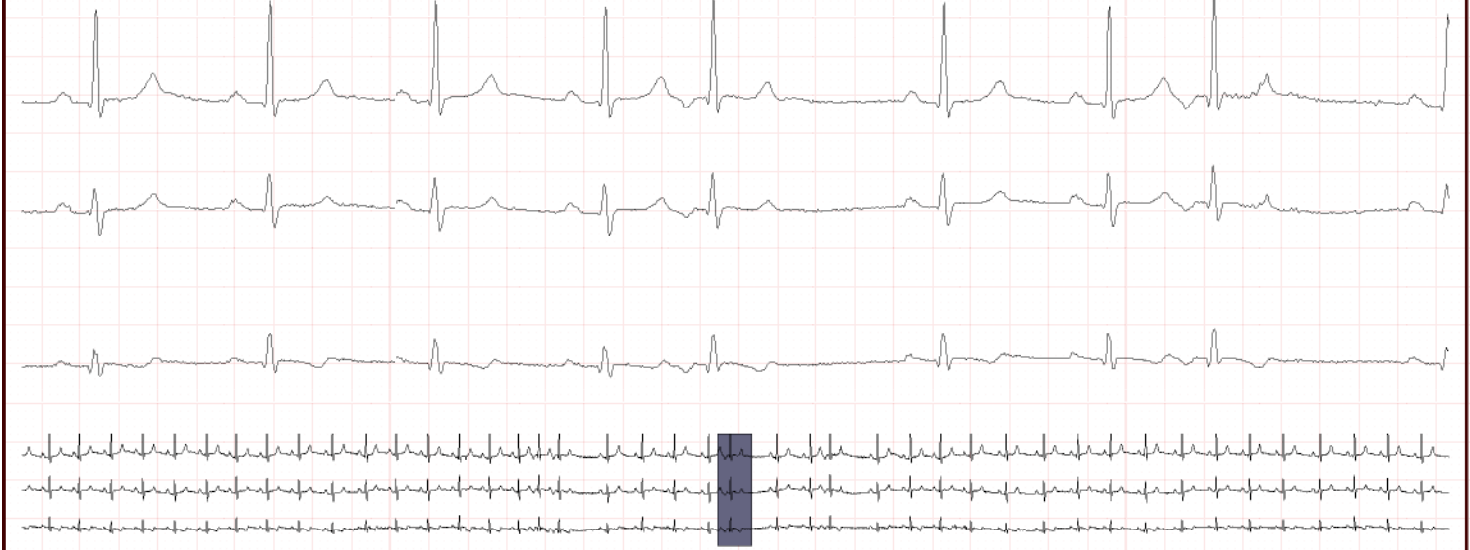

10:00:23 AM(2)

.94 .95 .90 **Fast Heart Beat, Dizzy, SOB** .91 .94 .93 **HR: 66** .93

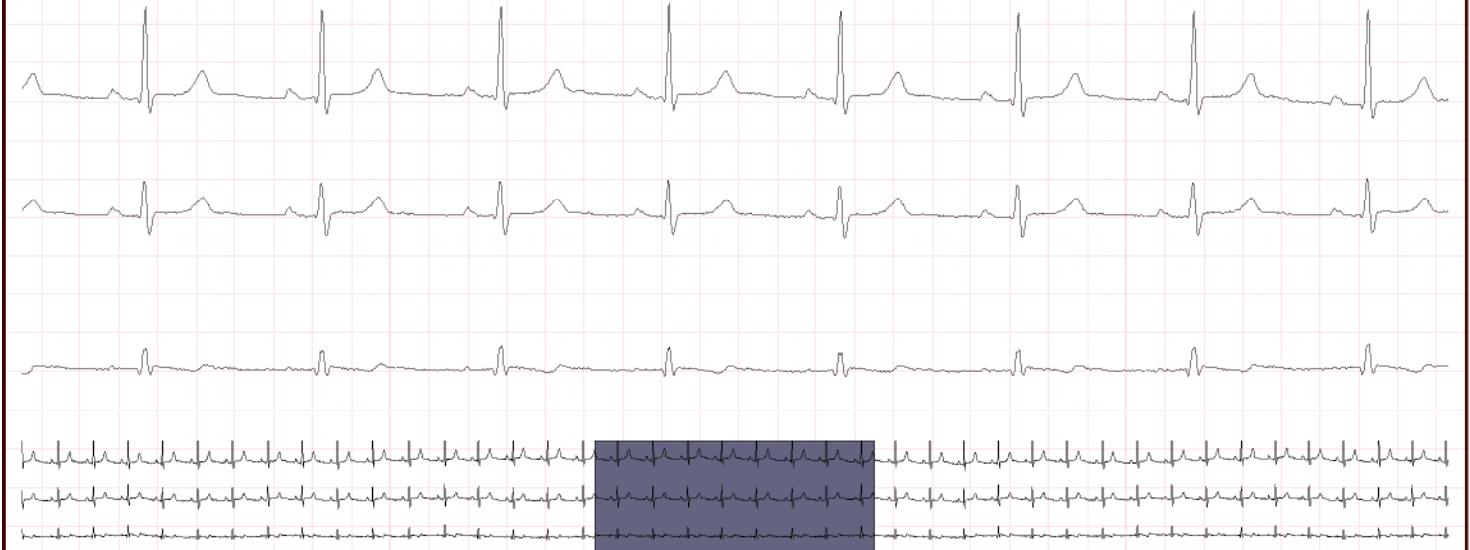

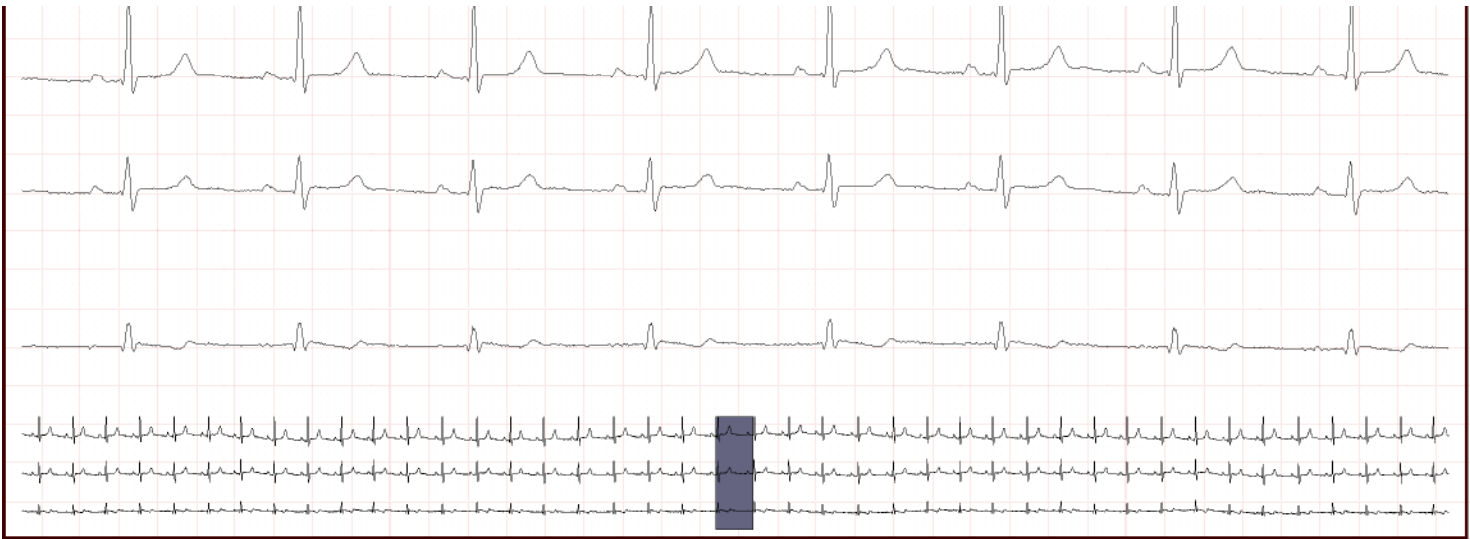

11:00:23 AM(2)

.91

.90

Fast Heart Beat, Dizzy, SOB

.91


.90

.89

.93

HR: 68

.93

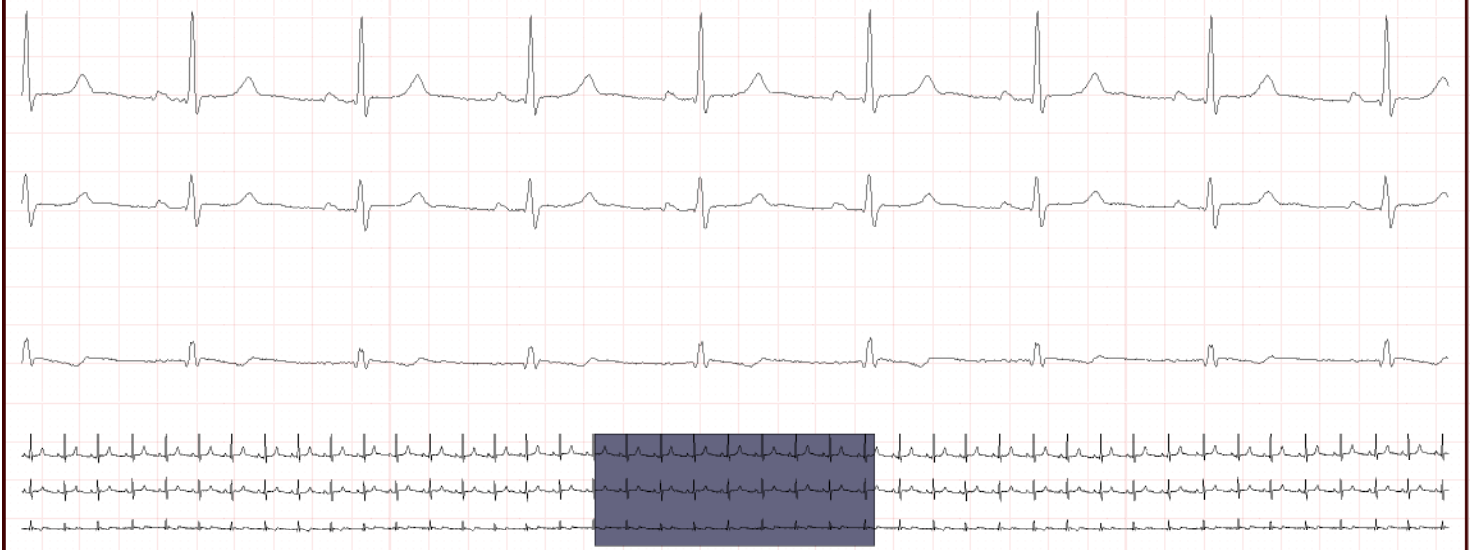

3:19:47 PM(2)

.96

.92

.96

Minimum HR


.97


.94

.98

HR: 63

.92

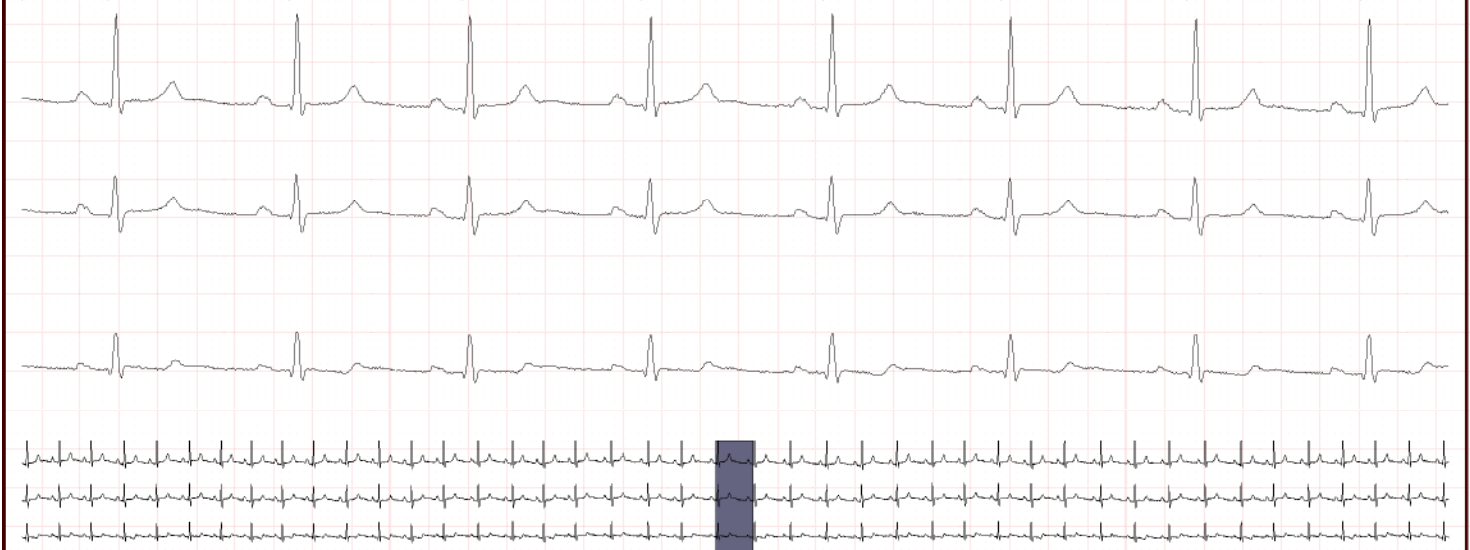


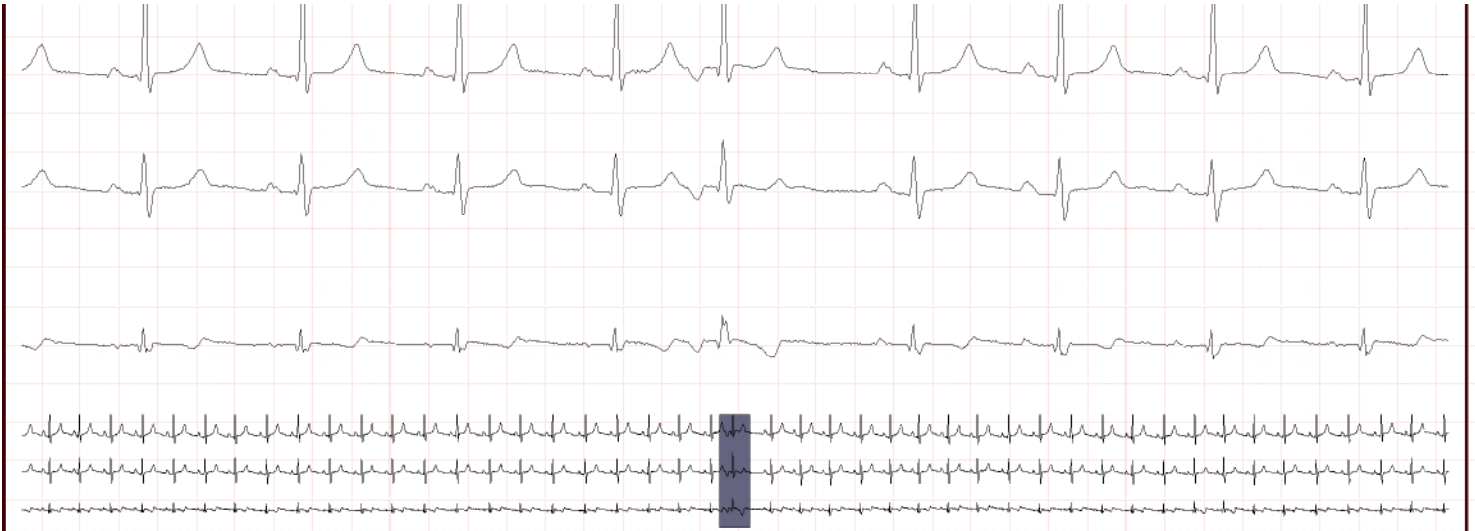

7:01:20 PM(2)

Maximum HR

HR: 133

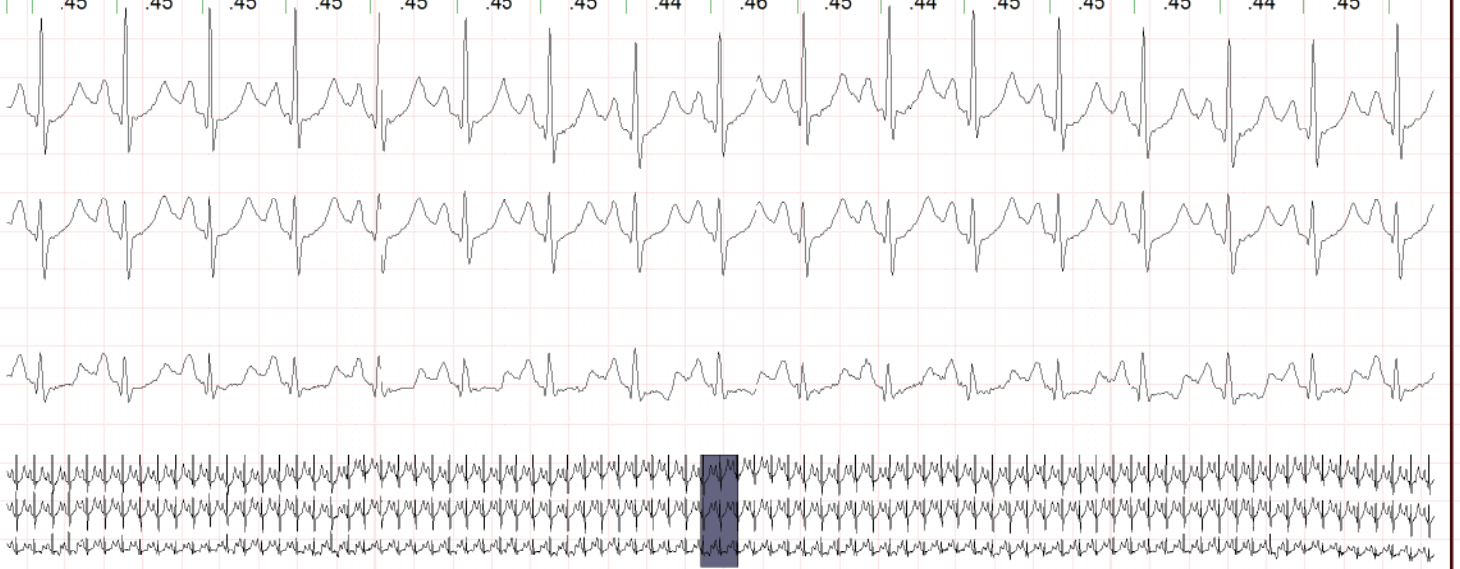


9:30:02 PM(2)

Sinus Rhythm


HR: 77



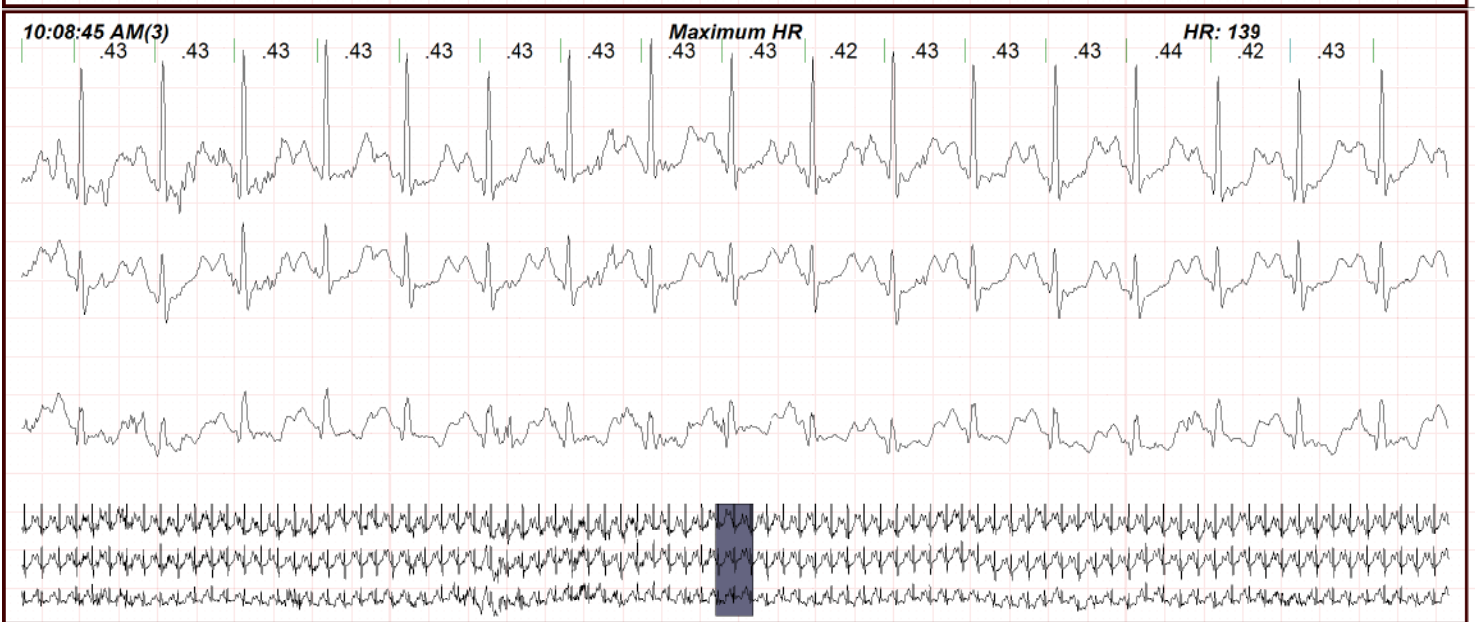


12:00 PM(1)	70	84	103	5026	0	0	0	0	0	0	0	59:34	0:25
2:00 PM(1)	81	96	124	5760	0	0	0	0	0	0	0	58:35	1:24
3:00 PM(1)	76	92	115	5499	0	0	0	0	0	0	0	59:36	0:23
4:00 PM(1)	74	90	119	5390	0	0	0	0	0	0	0	59:27	0:32
5:00 PM(1)	63	87	106	5178	0	0	0	0	0	0	0	58:59	1:00
6:00 PM(1)	70	88	116	5275	0	0	0	0	0	0	0	58:48	1:11
7:00 PM(1)	75	94	113	5638	0	0	0	0	0	0	0	59:11	0:48
8:00 PM(1)	68	93	117	5509	0	0	0	0	0	0	0	58:55	1:04
9:00 PM(1)	71	80	99	4774	0	0	0	0	0	0	0	59:48	0:11
10:00 PM(1)	68	74	102	4445	0	0	0	0	0	0	0	59:45	0:14
11:00 PM(1)	70	78	110	4698	0	0	0	0	0	0	0	59:45	0:14
12:00 AM(2)	71	90	117	5316	0	0	0	0	0	0	0	58:51	1:08
1:00 AM(2)	71	97	108	4574	0	0	0	0	0	0	0	59:14	0:45
2:00 AM(2)	68	77	95	4643	0	0	0	0	0	0	0	59:33	0:26
3:00 AM(2)	64	73	102	4340	0	0	0	0	0	0	0	59:53	0:06
4:00 AM(2)	63	72	91	4300	0	0	0	0	0	0	0	59:54	0:05
5:00 AM(2)	65	73	99	4352	0	0	0	0	0	0	0	59:57	0:02
6:00 AM(2)	64	71	106	4233	0	0	0	0	0	0	0	59:49	0:10
7:00 AM(2)	67	88	116	5198	0	0	0	0	0	0	0	59:05	0:54
8:00 AM(2)	69	80	117	4787	0	0	0	0	0	0	0	59:27	0:32
9:00 AM(2)	63	74	102	4363	0	0	0	21	5	0	0	59:39	0:20
10:00 AM(2)	63	69	91	4146	0	0	0	0	0	0	0	59:59	0:00
11:00 AM(2)	63	76	120	4470	0	0	0	0	0	0	0	59:38	0:21
12:00 PM(2)	68	88	121	5242	0	0	0	0	0	0	0	58:43	1:16
1:00 PM(2)	71	84	131	4941	0	0	0	0	0	0	0	59:33	0:26
2:00 PM(2)	63	89	96	4129	0	0	0	0	0	0	0	59:55	0:04
3:00 PM(2)	63	74	110	4409	0	0	0	0	0	0	0	59:44	0:15
4:00 PM(2)	66	73	103	4371	0	0	0	1	0	0	0	59:53	0:06
5:00 PM(2)	79	96	118	5660	0	0	0	0	0	0	0	58:44	1:15
6:00 PM(2)	74	102	121	6085	0	0	0	0	0	0	0	59:00	0:59
7:00 PM(2)	75	86	133	5111	0	0	0	0	0	0	0	58:54	1:05
8:00 PM(2)	73	99	134	5904	0	0	0	0	0	0	0	58:49	1:10
9:00 PM(2)	74	81	109	4852	0	0	0	0	0	0	0	59:50	0:09
10:00 PM(2)	71	77	106	4643	0	0	0	0	0	0	0	59:41	0:18
11:00 PM(2)	70	80	116	4760	0	0	0	0	0	0	0	59:41	0:18
12:00 AM(3)	68	74	97	4438	0	0	0	0	0	0	0	59:26	0:33
1:00 AM(3)	66	76	103	4526	0	0	0	0	0	0	0	59:40	0:19
2:00 AM(3)	68	75	103	4488	0	0	0	0	0	0	0	59:56	0:03
3:00 AM(3)	65	70	86	4178	0	0	0	0	0	0	0	59:59	0:00
4:00 AM(3)	63	69	108	4110	0	0	0	0	0	0	0	59:35	0:24
5:00 AM(3)	64	71	94	4251	0	0	0	0	0	0	0	59:49	0:10
6:00 AM(3)	63	76	98	4205	0	0	0	0	0	0	0	59:27	0:32
7:00 AM(3)		78	108	1176	0	0	0	0	0	0	0	11:34	48:25
8:00 AM(3)	72	98	132	5355	0	0	0	0	0	0	0	54:43	5:16
9:00 AM(3)	77	99	136	5961	0	0	0	0	0	0	0	59:04	0:55
10:00 AM(3)	79	91	139	5481	0	0	0	0	0	0	0	59:45	0:14
11:00 AM(3)	73	92	123	5456	0	0	0	0	0	0	0	59:03	0:56
12:00 PM(3)	77	96	125	3397	0	0	0	0	0	0	0	35:07	0:52
TOTALS	63	80	139	231034	0	0	0	22	5	0	0	46:40:58	1:19:01




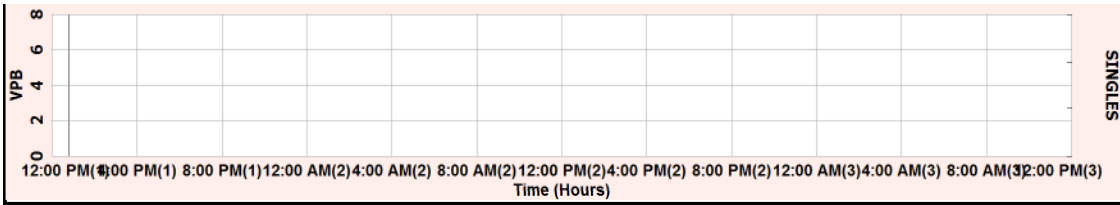


VPB Pair: 0

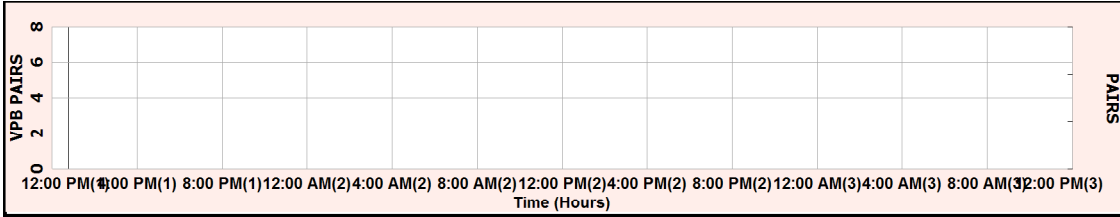



VTAC: 0

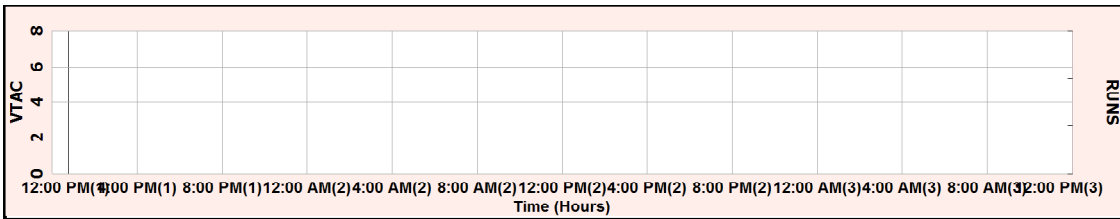


SVPB: 12

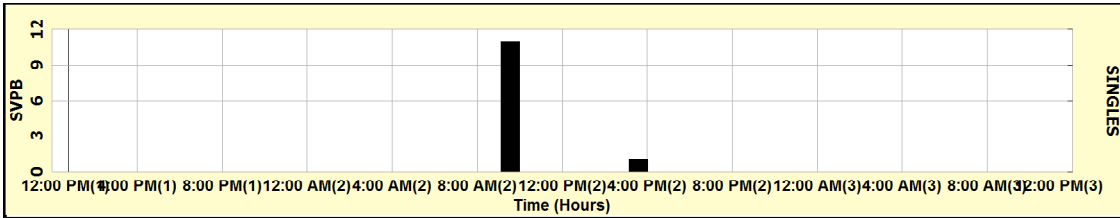


SVPB Pairs: 5

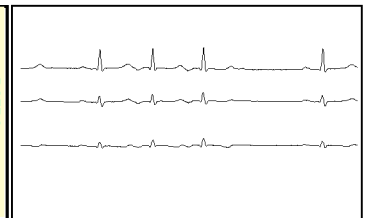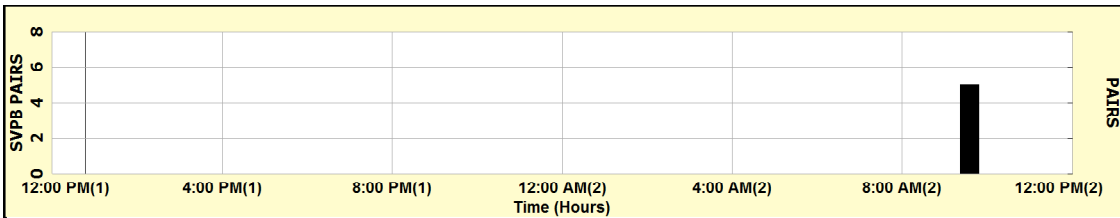

SVT: 0

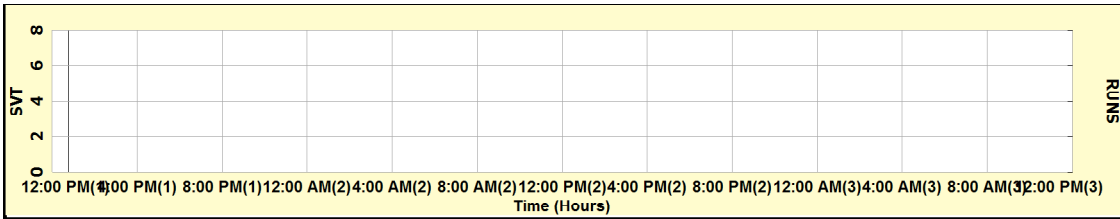

Pauses: 0

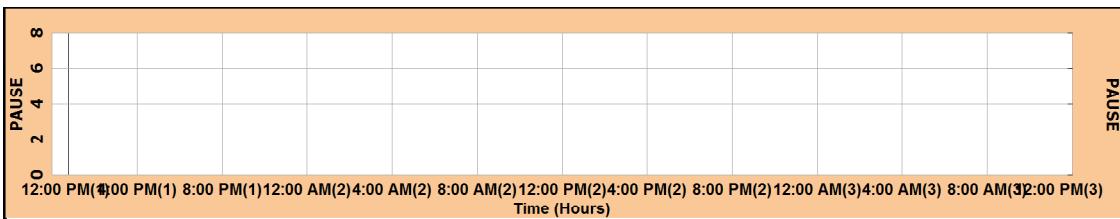

12:00 PM(1)	70	84	103	5026	0	0	0	0	0	0	59:34	0:25
2:00 PM(1)	81	96	124	5760	0	0	0	0	0	0	58:35	1:24
3:00 PM(1)	76	92	115	5499	0	0	0	0	0	0	59:36	0:23
4:00 PM(1)	74	90	119	5390	0	0	0	0	0	0	59:27	0:32
5:00 PM(1)	63	87	106	5178	0	0	0	0	0	0	58:59	1:00
6:00 PM(1)	70	88	116	5275	0	0	0	0	0	0	58:48	1:11
7:00 PM(1)	75	94	113	5638	0	0	0	0	0	0	59:11	0:48
8:00 PM(1)	68	93	117	5509	0	0	0	0	0	0	58:55	1:04
9:00 PM(1)	71	80	99	4774	0	0	0	0	0	0	59:48	0:11
10:00 PM(1)	68	74	102	4445	0	0	0	0	0	0	59:45	0:14
11:00 PM(1)	70	78	110	4698	0	0	0	0	0	0	59:45	0:14
12:00 AM(2)	71	90	117	5316	0	0	0	0	0	0	58:51	1:08
1:00 AM(2)	71	97	108	4574	0	0	0	0	0	0	59:14	0:45
2:00 AM(2)	68	77	95	4643	0	0	0	0	0	0	59:33	0:26
3:00 AM(2)	64	73	102	4340	0	0	0	0	0	0	59:53	0:06
4:00 AM(2)	63	72	91	4300	0	0	0	0	0	0	59:54	0:05
5:00 AM(2)	65	73	99	4352	0	0	0	0	0	0	59:57	0:02
6:00 AM(2)	64	71	106	4233	0	0	0	0	0	0	59:49	0:10
7:00 AM(2)	67	88	116	5198	0	0	0	0	0	0	59:05	0:54
8:00 AM(2)	69	80	117	4787	0	0	0	0	0	0	59:27	0:32
9:00 AM(2)	63	74	102	4363	21	11	5	0	0	0	59:39	0:20
10:00 AM(2)	63	69	91	4146	0	0	0	0	0	0	59:59	0:00
11:00 AM(2)	63	76	120	4470	0	0	0	0	0	0	59:38	0:21
12:00 PM(2)	68	88	121	5242	0	0	0	0	0	0	58:43	1:16
1:00 PM(2)	71	84	131	4941	0	0	0	0	0	0	59:33	0:26
2:00 PM(2)	63	89	96	4129	0	0	0	0	0	0	59:55	0:04
3:00 PM(2)	63	74	110	4409	0	0	0	0	0	0	59:44	0:15
4:00 PM(2)	66	73	103	4371	1	1	0	0	0	0	59:53	0:06
5:00 PM(2)	79	96	118	5660	0	0	0	0	0	0	58:44	1:15
6:00 PM(2)	74	102	121	6085	0	0	0	0	0	0	59:00	0:59
7:00 PM(2)	75	86	133	5111	0	0	0	0	0	0	58:54	1:05
8:00 PM(2)	73	99	134	5904	0	0	0	0	0	0	58:49	1:10
9:00 PM(2)	74	81	109	4852	0	0	0	0	0	0	59:50	0:09
10:00 PM(2)	71	77	106	4643	0	0	0	0	0	0	59:41	0:18
11:00 PM(2)	70	80	116	4760	0	0	0	0	0	0	59:41	0:18
12:00 AM(3)	68	74	97	4438	0	0	0	0	0	0	59:26	0:33
1:00 AM(3)	66	76	103	4526	0	0	0	0	0	0	59:40	0:19
2:00 AM(3)	68	75	103	4488	0	0	0	0	0	0	59:56	0:03
3:00 AM(3)	65	70	86	4178	0	0	0	0	0	0	59:59	0:00
4:00 AM(3)	63	69	108	4110	0	0	0	0	0	0	59:35	0:24
5:00 AM(3)	64	71	94	4251	0	0	0	0	0	0	59:49	0:10
6:00 AM(3)	63	76	98	4205	0	0	0	0	0	0	59:27	0:32
7:00 AM(3)		78	108	1176	0	0	0	0	0	0	11:34	48:25
8:00 AM(3)	72	98	132	5355	0	0	0	0	0	0	54:43	5:16
9:00 AM(3)	77	99	136	5961	0	0	0	0	0	0	59:04	0:55
10:00 AM(3)	79	91	139	5481	0	0	0	0	0	0	59:45	0:14
11:00 AM(3)	73	92	123	5456	0	0	0	0	0	0	59:03	0:56
12:00 PM(3)	77	96	125	3397	0	0	0	0	0	0	35:07	0:52
Totals	63	80	139	231034	22	12	5	0	0	0	46:40:58	1:19:01